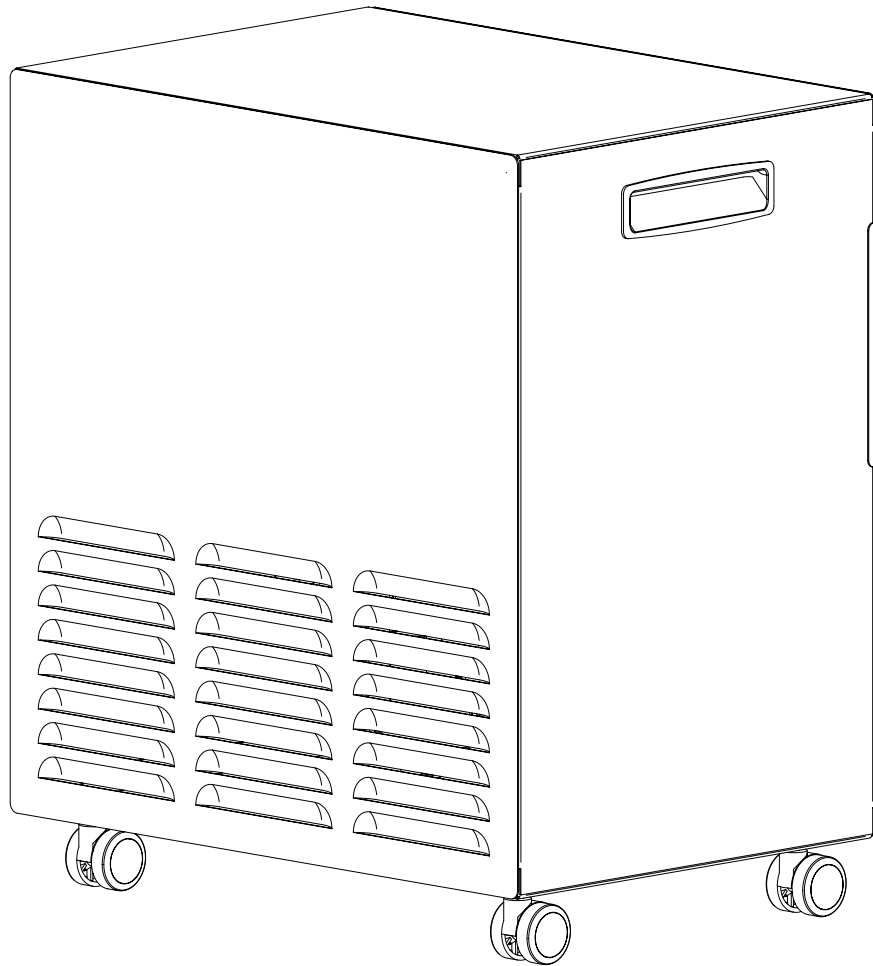
ActivePure®

GUARDIAN PRO

READ MANUAL CAREFULLY FOR PROPER USE AND OPERATION.

GENERAL INFORMATION

PRODUCT OVERVIEW

Your new ActivePure® Guardian Pro is designed to provide years of trouble-free, low maintenance operation. Be sure to read and follow all service procedures outlined in this owner's manual, and use only genuine ActivePure replacement parts.

CONTENTS

Product Overview	2
Specifications	2
Storage and Disposal	2
Important Safety Instructions	3
For Canadian Customers Only	4
Product Features	5
Operation	6
Replacement Parts & Service	9
Warranty Information	10
FCC Declaration & Responsible Party	11

STORAGE AND DISPOSAL

Do not dispose of this product as household waste. Electrical waste and electronic products should be appropriately recycled as required by practices established by your local authority.

SPECIFICATIONS

ActivePure Guardian Pro

- Model number: F203E
- Sales number: 00769

Technology:

- ActivePure® — (4) Cells

Electrical:

- Input Voltage: 120 V; 60 Hz
- Power Consumption: 216 W
- Complies to UL 507

Mechanical:

- Fuse: 2.5 A slowblow
- Airflow Rate: 300 CFM

Sound Level:

- 69 dB(A)

Weight & Dimensions:

- 27 lbs (12.25 kg)
- 13" W x 21.5" H x 18.5" D (33.02 cm x 54.61 cm x 46.99 cm)

Recommended Maximum Coverage in a Contiguous Space¹:

- Every facility is unique and proper configuration is an important as the technology. To validate coverage in your facility, speak to your sales representative or call customer service at 800-572-1446.

Warranty:

- Limited Three (3) Year Warranty

ActivePure Manufacturing, LLC reserves the right to change or modify any specification without notice.

If you have any questions concerning this, or any ActivePure product, contact ActivePure Customer Service at 800.572.1446 or email customerservice@ActivePure.com.

IMPORTANT SAFETY INSTRUCTIONS

CAUTION

- Risk of property damage, injury or death
- Any service must be performed by a qualified technician
- Risk of Sharp Edges Hazard
- Equipment sharp edges can cause injuries; avoid grasping equipment edges without protective gloves

WARNING

- Electrical Shock Hazard
- Can cause injury or death
- Disconnect all electrical power supplies before servicing
- When using an electrical appliance, basic precautions should always be taken
- To reduce risk of fire, electrical shock, or injury to persons, observe the following:

Read this manual in its entirety before using your ActivePure® Guardian Pro.

Keep out of the reach of children.

- Use only as described in this manual
- **DO NOT** use outdoors or on wet surfaces
- **DO NOT** allow to be used as a toy; Keep out of reach from children; close attention is necessary if operated near children
- **DO NOT** use with damaged cord or plug. If unit is not working as it should, has been dropped, damaged, left outdoors, or dropped into water, **DO NOT** use, and return it to a service center
- **DO NOT** pull or carry by cord, use cord as a handle, close door on cord, or pull cord around sharp edges or corners. Keep cord away from heated surfaces
- **DO NOT** unit to sit on cord
- **DO NOT** unplug by pulling on cord. To unplug, grasp the plug, not the cord
- **DO NOT** handle plug or unit with wet hands
- **DO NOT** allow foreign objects to enter the ventilation or exhaust openings
- Turn off all controls before unplugging
- To avoid electric shock and fire hazards, plug directly into an appropriate electrical outlet (see voltage on unit); Never use with an extension cord or power strip
- To reduce the risk of electrical shock, this unit has a grounded plug (with a third prong); This plug will fit in a grounded outlet only one way. If it does not fit, contact a qualified electrician to install the proper outlet. **DO NOT** change the plug in any way. **DO NOT** use adapters

SAVE THESE INSTRUCTIONS

FOR CANADIAN CUSTOMERS ONLY

ACTIVEPURE® GUARDIAN PRO

Sales No. : 00769

This UV device helps reduce the spread of viruses in the air.

Keep out of the reach of children.

Please read the label and owner's manual before using: www.activepure.com/manuals

PRECAUTIONS:

WARNING: UV radiation emitted from this device. Unintended use of the device, or damage to the housing, may result in exposure to ultraviolet radiation. Ultraviolet radiation may cause eye and skin irritation. Avoid exposing eyes and skin to ultraviolet radiation.

FIRST AID INSTRUCTIONS: Please contact your healthcare provider in the event of injury caused by this device.

DIRECTIONS FOR USE:

Place the unit in the center of the area to be treated for best results. Plug the cord into a working electrical outlet and rotate the knob to the right to operate the unit. Please see owner's manual for detailed instructions on using, handling, storing and disposing of the device, and procedures to mitigate any associated risks.

The use of this device is a supplement to and not a substitute for standard infection control practices; users must continue to follow all current infection control practices, including those related to the cleaning and disinfection of environmental surfaces.

FOR INQUIRIES CONTACT:

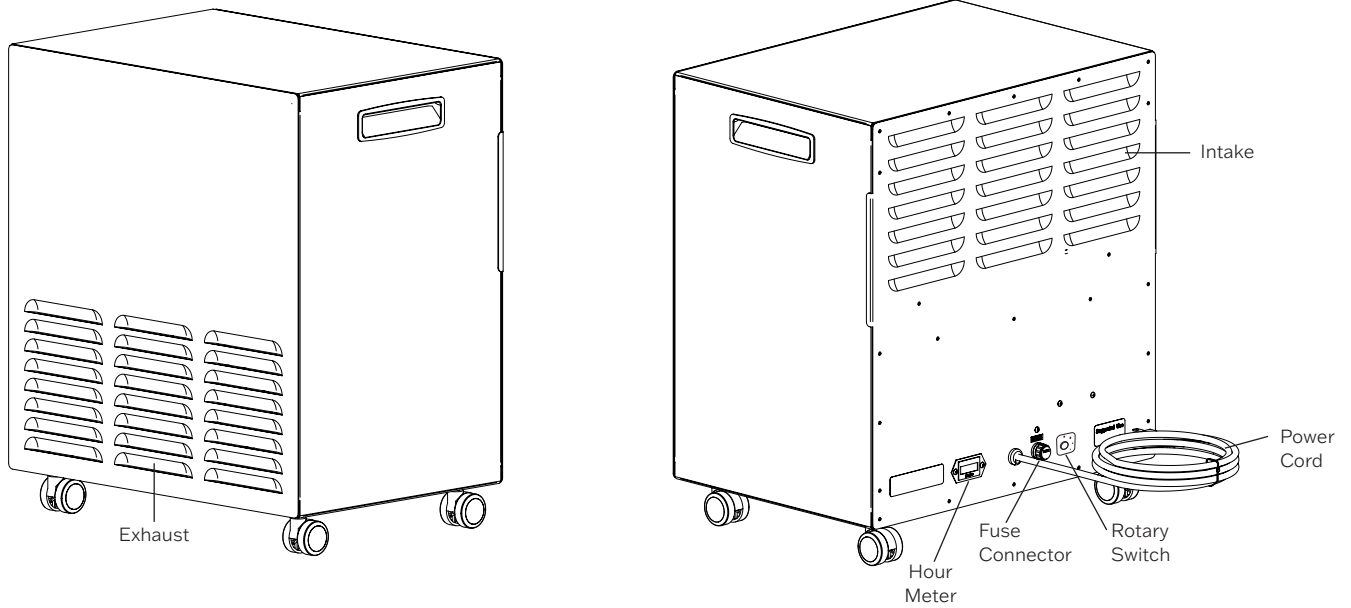
AERUS CANADA, INC.
Laird Business Park
3480 Laird Road, Units 2-5
Mississauga, Ontario, Canada, L5L 5Y4

W. Parrell, Manager
(905) 569-9399
wparrell@aerusonline.com

Aerus Canada, Inc. is affiliated with ActivePure Manufacturing, LLC
1300 Valley Drive, Bristol, VA 24201

PRODUCT FEATURES


It is important to be familiar with the parts of the ActivePure® Guardian Pro. If any parts are missing, contact your sales representative or ActivePure Customer Service.

OPERATION

The ActivePure Guardian Pro is only to be serviced by an authorized technician. This is a device intended for the reduction of viruses, bacteria, and other contaminants in large indoor spaces.

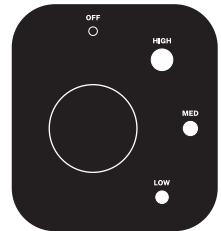1. Take your ActivePure Guardian Pro unit out of the shipping carton and remove all protective materials.
2. Place the unit in the center of the area to be treated for best results.
3. Plug the cord into a working electrical outlet and rotate the knob to the right to operate the unit.
4. Rotate knob to the high position first to power on the unit. Once the fan is moving at full speed, turn the knob to the desired speed.
5. When finished, rotate knob to the off position and unplug the unit.

NOTE: Periodic vacuuming of the filter screen may be required.

REPLACEMENT PARTS & SERVICE

REPLACEMENT PARTS (QTY)	PART NUMBER
ActivePure® Cell	49945

To order replacement parts, contact your sales representative or ActivePure Customer Service for further assistance at 800.572.1446 or email customerservice@ActivePure.com.

WARRANTY INFORMATION

Limited Three (3) Year Warranty

To register your ActivePure® Guardian Pro unit, visit www.activepure.com/warranty.

Customer should not repackage and ship the ActivePure Guardian Pro unit due to possibility of irreparable damage.

WHAT IS COVERED BY THIS WARRANTY

We warrant the ActivePure Guardian Pro to the consumer, subject to the conditions below, against defects in workmanship or material, provided that the products are returned to a service location within the following time periods:

- ActivePure Guardian Pro within three (3) years from date of purchase
- ActivePure Cell within one (1) year from date of purchase

INSTALLATION AND MAINTENANCE REQUIREMENTS

Failure to use and maintain your ActivePure Guardian Pro in accordance with your Owner's Manual will void this warranty. Servicing the ActivePure Guardian Pro by parties other than our authorized representatives and/or using parts other than genuine ActivePure parts will also void this warranty.

- Replacing the ActivePure Cell every one (1) year with a genuine ActivePure part (proof of change may be required)

HOW TO OBTAIN WARRANTY SERVICE

Customers must contact their sales representative or ActivePure Customer Service and provide proof of purchase within the above time periods. We will repair or replace and return the product, without charge and within a reasonable period of time, subject to the conditions in the above paragraphs, if its examination shall disclose any part to be defective in workmanship or material. If we, in our discretion, are unable to repair the product after a reasonable number of attempts, we will provide either a refund of the purchase price or a replacement unit, at the company's option.

WHAT IS NOT COVERED BY THIS WARRANTY

This product is intended for household and commercial use. Ordinary wear and tear shall not be considered a defect in workmanship or material. These warranties do not apply for loss or damage caused by accident, fire, abuse, misuse, modification, or misapplication.

EXCLUSION OF OTHER WARRANTIES AND CONDITIONS

EXCEPT AS PROVIDED HEREIN, AND TO THE EXTENT CONSISTENT WITH APPLICABLE LAWS, ACTIVEPURE MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND. ALL OTHER WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, ARE HEREBY EXPRESSLY DISCLAIMED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

UNAUTHORIZED CHANNELS & MISSING SERIAL NUMBERS

If a valid serial number is missing from the product, the warranty will be voided. Warranties are voided if a product is purchased through unauthorized channels, this includes websites that are not authorized to sell our products OR to

6

use ActivePure trademarked names, images and logos as well as Internet auction sites (e.g. ebay and Craigslist). The only approved Internet presence for ActivePure products is www.activepure.com or www.activepuremedical.com. To confirm warranty coverage prior to purchasing a product, contact ActivePure Customer Service at 800.572.1446 with the serial number located on the back of the unit.

LIMITATION OF LIABILITY FOR SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES

TO THE EXTENT CONSISTENT WITH APPLICABLE LAWS, ACTIVEPURE SHALL NOT IN ANY CASE BE LIABLE FOR SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING FROM BREACH OF EXPRESS OR IMPLIED WARRANTIES, CONDITIONS, GUARANTEES OR REPRESENTATIONS, BREACH OF CONTRACT, NEGLIGENCE OR ANY OTHER LEGAL THEORY. Such excluded damages include, but are not limited to, loss of profits or revenue and loss of the use of the products.

FOR U.S. APPLICATION ONLY

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state. Some states do not allow limitations on warranties, or on remedies for breach. In such states, the above limitations may not apply to you.

FOR CANADIAN APPLICATION ONLY

Exclusion of Subsequent Owners: Except as otherwise required by applicable legislation, this warranty is not transferable. This warranty gives you specific legal rights and you may also have other rights which vary from province to province. Some provinces and territories do not allow limitations on warranties, or on remedies for breach. In such provinces or territories, the above limitations may not apply to you. If any provision of this warranty or part thereof is held by a court of competent jurisdiction to be invalid, illegal or unenforceable, the validity, legality and enforceability of the remaining provisions or parts thereof will not in any way be affected or impaired within the jurisdiction of that court. This entire warranty shall continue to be valid, legal and enforceable in any jurisdiction where a similar determination has not been made.

This warranty is provided by:
ActivePure Manufacturing, LLC
1300 Valley Drive
Bristol, VA 24201

SERVICE

Every effort is made to ensure customers receive an up-to-date instructions manual on the use of our products; however, from time to time, modifications to our products may without notice make the information contained herein subject to alteration. For the latest information, please visit our website at www.activepure.com or www.activepure.com/manuals.

FCC DECLARATION & RESPONSIBLE PARTY

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant of part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the Independent Contractor for help.

FCC Declaration of Conformity

Name: ActivePure® Guardian Pro

Model: F203E

Produced by: ActivePure Manufacturing, LLC

This device complies with Part 15 of the FCC Rules.

Responsible Party

ActivePure Manufacturing, LLC

1300 Valley Drive

Bristol, VA 24201

Ph: 800.572.1446

Signature: 

Printed Name: Andrew Eide

Title: Vice President of Product Development
and Manufacturing

Date: 01/11/22

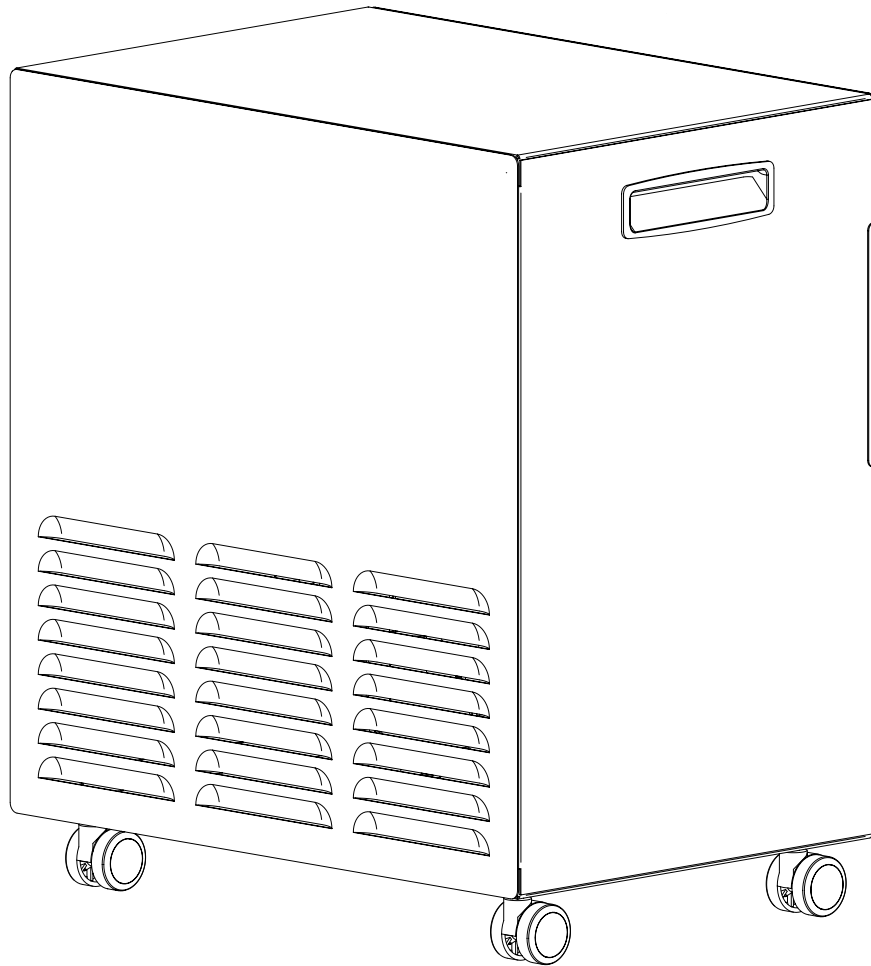
For information regarding the use of this product, please call Customer Service.

1.800.572.1446 or email at customerservice@ActivePure.com

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Fabricant: ActivePure Manufacturing, LLC